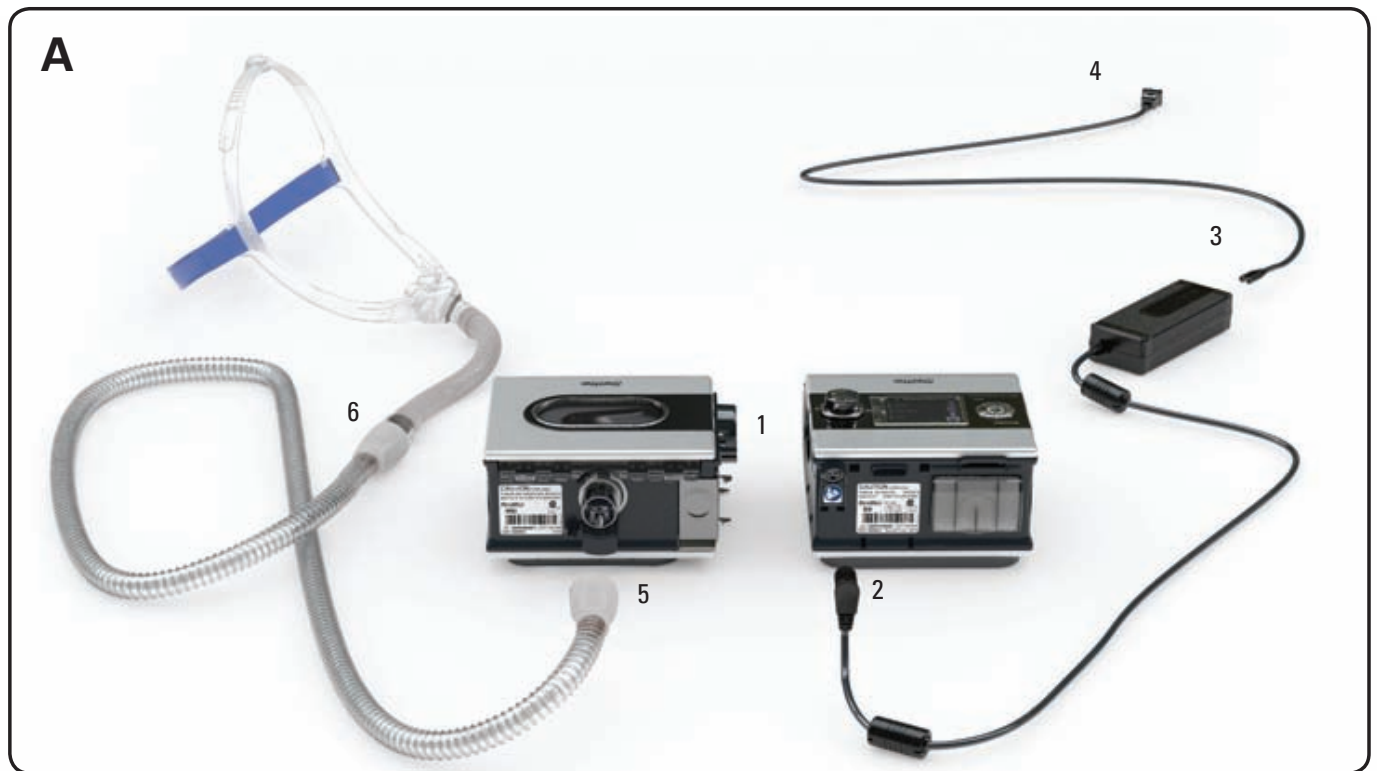
RESMED

H5i™

HEATED HUMIDIFIER

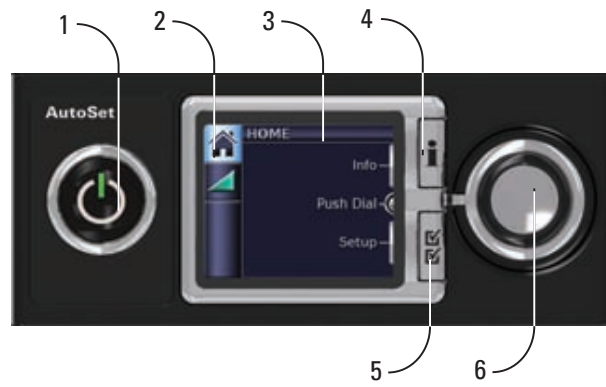
User Guide

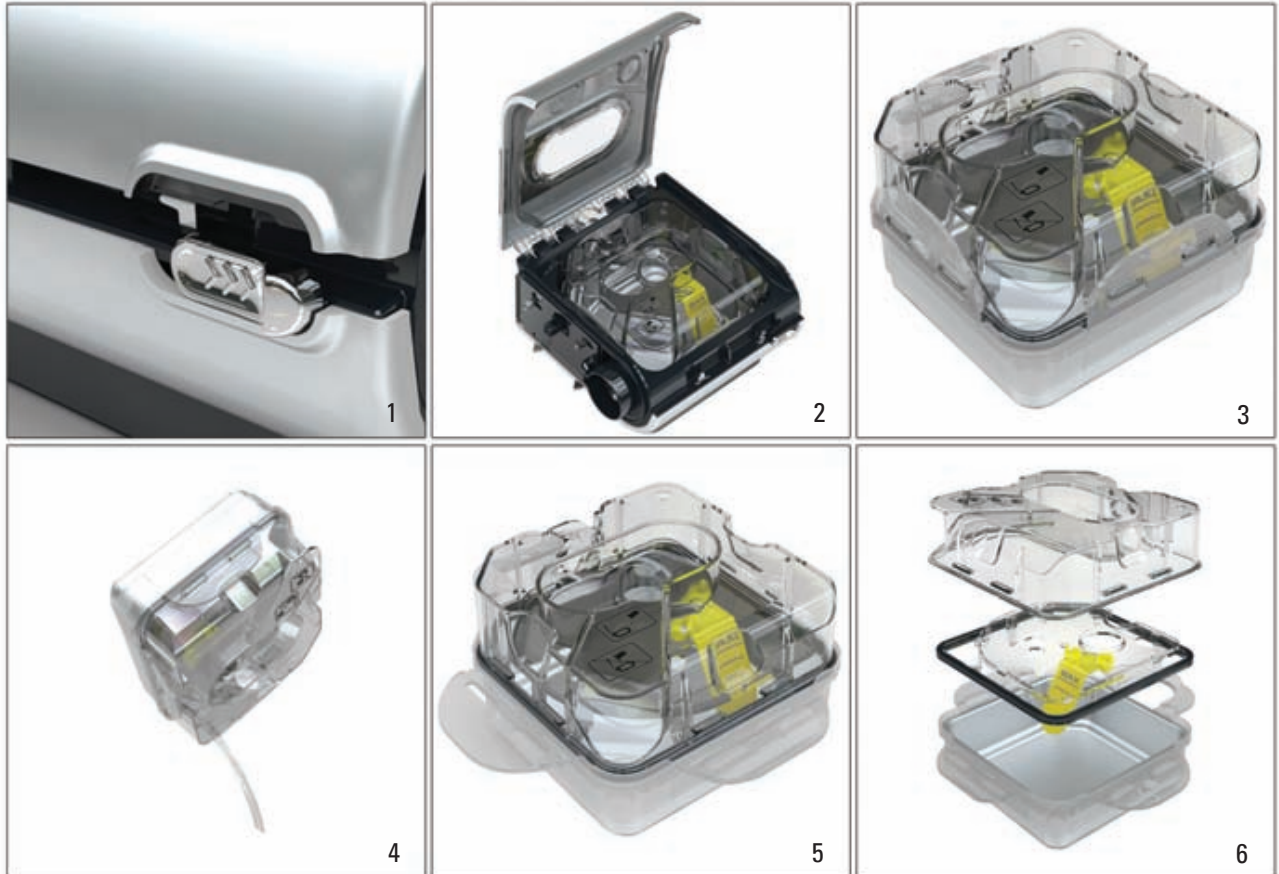
English • Deutsch • Français • Italiano • Español • Português • Nederlands • Ελληνικά • Polski • Magyar •
Русский • Türkçe • Český • Slovensky

B

C**D**

E

Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. **Distributed by:** ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See www.resmed.com for other ResMed locations worldwide.

S9: Protected by patents: AU 691200, AU 697652, AU 702820, AU 709279, AU 724589, AU 730844, AU 736723, AU 750095, AU 750761, AU 764761, AU 779327, EP 0651971, EP 0661071, EP 0920845, EP 0927538, EP 0934723, EP 1028769, EP 1126893, EP 1144036, EP 1502618, JP 3580776, JP 3778797, NZ 504595, US 5199424, US 5245995, US 5522382, US 5704345, US 6029665, US 6138675, US 6240921, US 6363933, US 6367474, US 6425395, US 6502572, US 6591834, US 6675797, US 6745768, US 6817361, US 6988498, US 7040317, US 7100608, US 7320320, US 7537010.

Other patents pending. Design registrations pending. S9 is a trademark of ResMed Ltd and is registered in U.S. Patent and Trademark Office.

H5i: Protected by AU 2002233025, CN ZL02804936.5, HK 1065483, NZ 527088, US 6935337. Other patents pending. Designs pending.

© 2009 ResMed Ltd. 368148/1 09 11

H5i indications for use

The H5i is indicated for the humidification of the air delivered from a CPAP device. The H5i is for use only as recommended by a physician. The H5i is intended for single patient re-use in the home environment and re-use in a hospital/institutional environment.

Contraindications

The H5i is contraindicated for use with patients whose upper (supraglottic) airway has been bypassed. Please refer to the S9 Series User Guide for contraindications associated with CPAP therapy.

Setup

Refer to illustration A.

1. Align your H5i with your S9 and push them together until they click into place.
2. Connect the DC plug of the power supply unit to the rear of the S9.
3. Connect the power cord to the power supply unit.
4. Plug the other end of the power cord into the power outlet.
5. Connect one end of the air tubing firmly onto the air outlet.
6. Connect the assembled mask system to the free end of the air tubing.

Filling the water tub

Refer to illustration B.

1. Slide the latch and lift open the flip lid.
2. Remove the water tub.
3. Fill the water tub (through the centre hole) with water up to the maximum water level mark (380 ml).
4. Return the water tub to the H5i.
5. Close the flip lid ensuring that it clicks into place.

Control panel

Refer to illustration C.

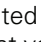
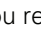
The control panel of your S9 devices includes the following items:

1. Start/Stop button: Starts or stops treatment
2. Home menu
3. LCD screen
4. Info menu: Allows you to view your sleep statistics
5. Setup menu: Allows you to make changes to settings
6. Push dial: Turning the dial allows you to scroll through the menu and change settings. Pushing the dial allows you to enter into a menu and confirm your choice.

Adjusting humidity level

Refer to illustration D.

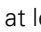
Ranging from OFF to 6, you can adjust the humidity level at any time to find the setting that is most comfortable for you. To adjust your level of humidity:

1. From your HOME screen, turn  (Push Dial) until the HUMIDITY LEVEL screen is displayed in blue.
2. Press . The selection is highlighted in orange.
3. Turn  until the humidity level that you require is displayed.
4. Press  to confirm your choice.

Warming the H5i

The warm-up feature is used to pre-heat the water before starting treatment.

To start warming:

1. Navigate to the humidity level icon.
2. Push and hold  for three seconds. The Warming Up status bar is displayed. You can stop warm-up at any time by pressing  for at least three seconds. You can start treatment at any time by pressing  (Start/Stop).

Note: In order to assist the heater plate in cooling, your S9 device will continue to blow air for up to an hour after treatment has stopped. However, you can unplug the device from the power outlet at any time and allow the heater plate to cool without air flow.

Getting started

1. Make sure your S9 and H5i are connected and the water tub is filled to the maximum water level mark.
2. Adjust the humidification level if required.
3. Fit your mask as described in your mask user guide.
4. To start therapy, simply breathe into the mask and/or press .
5. Lie down and arrange the air tubing so that it is free to move if you turn in your sleep.
6. To stop treatment at any time, remove your mask and/or press .

Notes:

- If your clinician has enabled SmartStart your device will start automatically when you breathe into the mask and stop automatically when you remove the mask.
- Once therapy has started the HUMIDITY LEVEL screen is displayed.

Cleaning and maintenance

Disassembling the H5i

Refer to illustration E.

1. Slide the latch.
2. Lift open the flip lid.
3. Remove the water tub.
4. Discard any excess water from the water tub.
5. Unclip all four side latches.
6. Pull apart the tub lid, plate and base.

Daily:

1. Wash the disassembled tub lid, plate and base in warm water using a mild detergent.
2. Rinse thoroughly in clean water and allow them to dry away from direct sunlight.

Note: The disassembled tub lid, plate and base may also be washed in a dishwasher on the delicate or glassware cycle (top shelf only).

Monthly:

1. Peel the flip lid seal from the flip lid and wash it in warm water using a mild detergent.
2. Wipe the exterior of the H5i with a damp cloth and mild detergent.

Reassembling the H5i:

1. Place the plate back onto the base ensuring that the maximum water level mark faces up.
2. Place the tub lid back onto the plate/base ensuring that the centre holes are aligned.
3. Clip all four side latches.
4. Fill the water tub and return the water tub to the H5i.
5. Close the flip lid ensuring that it clicks into place.

Troubleshooting

If there is a problem, try the following suggestions. If the problem cannot be solved, contact your equipment supplier or ResMed. Do not attempt to open these devices.

Problem/Possible cause	Solution
No display	
Power is not connected.	Ensure the power cable is connected and the power outlet (if available) is on.
The S9 and H5i are not connected correctly.	Ensure that the S9 and H5i are securely attached.
Insufficient air delivered from the device	
The S9 is not working correctly.	See the Troubleshooting section of the S9 Series User Guide.
Air tubing is not connected properly.	Check air tubing.
Air tubing is blocked, pinched or punctured.	Unblock or free the air tubing. Check the air tubing for punctures.
The H5i cover is not latched correctly.	Ensure the cover is latched onto the docking station correctly.
The H5i cover seal is not fitted correctly.	Ensure the cover seal is facing the right way up and fitted securely.
Incorrect air tubing selected.	If you are using the SlimLine or Standard air tubing ensure that you have the correct air tubing selected via the menu.
Displays error message: Heated tube fault, replace tube	
Device has been left in a hot environment.	Allow to cool before re-use. Disconnect the power cord and then reconnect it to restart the device.
There is a fault in your ClimateLine air tubing.	Discontinue using your ClimateLine air tubing and contact your clinician/service provider. Use Standard or SlimLine tubing in the interim.
Displays error message: Humidifier fault, replace humidifier	
There is a fault in your H5i heated humidifier.	Discontinue using your H5i and contact your clinician/service provider.
Device has been left in a hot environment.	Allow to cool before re-use. Disconnect the power cord and then reconnect it to restart the device.
Displays error message: Tube blocked, please check your tube	
Air tubing is blocked.	Check your air tubing and remove any blockages.

Problem/Possible cause**Solution****Water splashing on your face from the H5i**

The water tub is overfilled. Check that the water level is below the maximum water level mark.
 Condensation is forming in the air tube and mask. Turn the H5i or relative humidity setting down via the menu.

Leaking water tub

The water tub may be damaged or cracked. Contact your service provider for a replacement.
 The cleanable water tub is not assembled correctly. Check for damage and reassemble the cleanable water tub correctly.

Air feels too warm/cold in the mask

The temperature of the ClimateLine air tubing is set too high/low. Turn up/down the heated tubing temperature via the menu.

Technical specifications

Maximum heater plate temperature	65°C
Temperature cut-out	74°C
Maximum gas temperature	≤41°C
Nominal dimensions (L x W x H)	Docking station and water tub: 153 mm x 145 mm x 86 mm
Weight (Water tub)	Docking station and unfilled water tub 0.67 kg
Weight (Cleanable water tub)	Docking station and unfilled water tub 0.77 kg
Water capacity	To maximum fill line 380 mL
Docking station	Flame retardant engineering thermoplastic, aluminium
Cleanable water tub	Injection molded plastic, stainless steel and silicone seal
Water tub	Injection molded plastic, aluminium and thermoplastic elastomer
Operating temperature	+5°C to +35°C
Operating humidity	10–95% non-condensing
Storage and transport temperature	-20°C to +60°C
Storage and transport humidity	10–95% non-condensing
ClimateLine air tubing	Flexible plastic and electrical components, 1.98 m, 15 mm inner diameter
Heated tubing temperature cut-out	≤ 41°C
IEC 60601-1 classification	Class II (double insulation), Type BF
Electromagnetic compatibility	Please refer to the S9 Information Guide for details

Notes:

- *The manufacturer reserves the right to change these specifications without notice.*
- *The temperature and relative humidity settings displayed for ClimateLine are not measured values.*
- *Check with your clinician/service provider before using the SlimLine air tubing with devices other than the S9 or H5i.*

Humidifier performance

The following settings have been tested at 22°C, 40% RH ambient conditions:

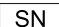
CPAP mask pressure cm H ₂ O	RH output %		Nominal system output AH ^a , BTPS ^b	
	Setting 3	Setting 6	Setting 3	Setting 6
4	100	100	21	27
10	100	100	16	23
20	100	100	14	17

a. AH- Absolute Humidity in mg/L.

b. BTPS- Body Temperature Pressure Saturated.

Symbols which appear on these devices

The following symbols may appear on your H5i, ClimateLine or packaging.

 Caution;
  Follow instructions for use;
  Drip proof;
  Manufacturer;
  European RoHS;
  LOT Lot number;
  REF Part number;
  SN Serial number;
  Not for use on more than one patient;
  MAX Maximum water level;
  Disinfectable up to 93°C;
  China pollution control logo 1;
  China pollution control logo 2;
  Lock/unlock;
  Remove tub to fill;

Environmental information

WEEE 2002/96/EC is a European Directive that requires the proper disposal of electrical and electronic equipment. These devices should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment.

If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to www.resmed.com/environment.

Servicing

The H5i device is intended to provide safe and reliable operation when operated in accordance with the instructions provided by ResMed. ResMed recommends that the H5i be inspected and serviced by an authorised ResMed Service Centre if there is any sign of wear or concern with device function. Otherwise, service and inspection of the devices generally should not be required during the five year design life of the device.

Limited warranty

ResMed Ltd (hereafter 'ResMed') warrants that your ResMed product shall be free from defects in material and workmanship from the date of purchase for the period specified below.

Product	Warranty period
<ul style="list-style-type: none">Mask systems (including mask frame, cushion, headgear and tubing) – excluding single-use devicesAccessories – excluding single-use devicesFlex-type finger pulse sensorsHumidifier water tubs	90 days
<ul style="list-style-type: none">Batteries for use in ResMed internal and external battery systems	6 months
<ul style="list-style-type: none">Clip-type finger pulse sensorsCPAP and bilevel device data modulesOximeters and CPAP and bilevel device oximeter adaptersHumidifiers and humidifier cleanable water tubsTitration control devices	1 year
<ul style="list-style-type: none">CPAP, bilevel and ventilation devices (including external power supply units)Battery accessoriesPortable diagnostic/screening devices	2 years

This warranty is only available to the initial consumer. It is not transferable.

If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This limited warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organisation that has not been expressly authorised by ResMed to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; and d) any damage caused by water being spilled on or into an electronic device.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty replaces all other expressed or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have resulted from the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

WARNINGS

- Read the entire manual before using these devices.
- Use the device only for its intended use as described in this manual.
- Do not open the H5i. There are no user serviceable parts inside. Repairs and servicing should only be performed by an authorised ResMed service agent.
- Beware of electrocution. Do not immerse the H5i, docking station or power cord in water. Always unplug the device before cleaning and make sure it is dry before plugging it back in.
- Do not use bleach, chlorine, alcohol, or aromatic-based solutions, moisturising or antibacterial soaps or scented oils to clean the H5i or air tubing. These solutions may cause damage and reduce the life of these products.
- Follow all precautions when using supplemental oxygen.
- Explosion hazard – do not use in the vicinity of flammable anaesthetics.
- Only use the ResMed 90W power supply units.
- Make sure the power cord and plug are in good condition and the equipment is not damaged.
- Keep the power cord away from hot surfaces.
- Do not drop or insert any object into any tubing or opening.
- Do not leave long lengths of air tubing around the top of your bed. It could twist around your head or neck while you are sleeping.
- Advice contained in this manual should not supersede instructions given by the prescribing physician.
- The H5i is designed only for use with specific ResMed devices. Do not use the H5i with any other CPAP device.
- Do not operate the H5i if it is not working properly or if any part of the CPAP device or H5i has been dropped or damaged.
- The H5i should only be used with tubing or accessories recommended by ResMed. Connection of other delivery tubes or accessories could result in injury, or damage to the device.
- Water must be prevented from entering your CPAP device.
- Do not overfill the water tub as water may enter the device and air tubing.
- Take care when handling your H5i as the water/water tub may be hot. Allow 10 minutes for the heater plate and any excess water to cool.
- The H5i docking station should only be connected or disconnected when the water tub is empty.
- Make sure that the water tub is empty before transporting the H5i.
- Do not operate the H5i on an aircraft as water may enter the CPAP device and air tubing during turbulence.
- Always place the H5i on a level surface below the level of the user to prevent the mask and tubing from filling with water.
- The H5i should be used with caution for users who are unable to remove the mask for themselves.
- If liquids are inadvertently spilled into or on the H5i docking station, unplug the device from the power outlet. Disconnect the docking station from the CPAP device and allow the docking station to drain and dry before re-using.
- If damage to the air tubing results in air leaking, stop using the air tubing.
- Do not wash the air tubing in a washing machine or dishwasher.
- Avoid covering the ClimateLine air tubing with materials such as sheets or blankets.
- The ClimateLine is designed only for use with the H5i.
- ClimateLine is not compatible with supplemental oxygen.

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