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Deutsch

12. After cleaning, perform a function check (see "6. Function check").
13. **We recommend changing the housing parts every 3 months.**

### 5.3. Disinfecting

You can disinfect the top and bottom parts of the bacteria filter. Follow the instructions for use for the disinfectant you are using.

We recommend GIGASEPT FF. You are recommended to wear suitable gloves (e.g. household or disposable gloves) during disinfection procedures.

- Procedure is as described in the section entitled „5.2 Cleaning“.
- In deviation from this procedure, the bacteria filter must be thoroughly rinsed with distilled water and then dried after being disinfected.

### 5.4. Sterilizing

The housing parts can be autoclaved up to 30x at 134 °C in devices to EN 285.

Minimum retention time: 3 min.  
Maximum retention time: 18 min.

- Procedure is as described in the section entitled „5.2 Cleaning“.
- In deviation from this procedure, the bacteria filter should not be rinsed with clean water following sterilization.

### 5.5. New patients

- The particle filter must always be changed before each new patient.
- To do so, proceed according to the section entitled „4.2 After use“.

## 6. Function check

### 6.1. Critical times

Perform a function check at least once a month. If you find faults during the function check you may not use the bacteria filter again until the faults have been eliminated.

### 6.2. Execution

- Subject the plastic housing and sealing element 4 to a visual inspection: if the bacteria filter is torn or otherwise damaged, it must be replaced completely.

## 7. Faults and how to eliminate them

Fault	Cause of fault	Eliminating fault
Bacteria filter making loud noises.	Housing not correctly screwed together. Cracks in bacteria filter housing	Screw the bacteria filter in again as described in the section entitled „3.1 Assembly“. Replace the bacteria filter.
No pressure or incorrect pressure building up at the mask.	Water in pressure measuring duct No mains voltage at the therapy device	Dry pressure measuring duct (5. Cleaning, disinfecting and sterilizing). Check that mains connection cable is securely attached. If appropriate, check function of socket by connecting another device (e.g. lamp) to it.

## 8. Maintenance

The bacteria filter requires no maintenance.

If you find faulty parts during the function check, please replace them.

### 8.1. Disposal

If the bacteria filter can no longer be used, please dispose of it via your hospital's waste disposal service or via domestic waste.

## 9. Scope of supply

### 9.1. Standard scope of supply

Bacteria filter, complete consisting of:	WM 24148
bacteria filter, assembled	WM 24639
kit, 10 particle filters	WM 15584
instructions for use D, GB, F	WM 16790

### 9.2. Spare parts

Kit I particle filters (10 x WM 24147)	WM 15584
Kit II particle filters (100 x WM 24147)	WM 15576
Kit III particle filters (500 x WM 24147)	WM 15577

## 10. Technical data

Product class to 93/42/EEC	IIa
Dimensions Ø x L in cm	7.4 x 9.8
Weight	approx. 51 g
Temperature range	
- operation	+5 °C to +40 °C
- storage	-20 °C to +70 °C
Permitted humidity for operation and storage	≤ 95 % r.h. (no condensation)
Ambient pressure range	700 to 1060 mbar
Increase in noise level	
1 m from the device in the patient's position as per EN ISO 17510	max. 0,5 dB(A)
V <sub>f</sub>	≤ 1,5 l
Max. permitted flow (flowing off freely)	300 l/min.
Internal volume of the bacteria filter	85 ml
Therapy pressure range	3 to 35 mbar
Filtration efficacy checked as per DIN EN 13328-1	
Usage time particle filter	≤ 24 h
Material for housing	PC
Housing can be autoclaved in	
devices to EN 285	134 °C

We reserve the right to make design modifications.

## 11. Warranty

Weinmann provides a warranty of 3 months from the date of purchase that the product is free of defects if used in accordance with purpose.

A condition for claims under warranty is submission of a proof of purchase showing the sales source and date of purchase.

We will not provide a warranty in the event of:

- the instructions for use not being followed
- operating faults
- incorrect use or treatment
- failure to use original spare parts
- third-party intervention in the device for repair purposes by non-authorized persons
- force majeure
- transport damage as a result of incorrect packing when returning devices
- ordinary wear and tear.

Weinmann will not be held liable for consequential damage unless due to deliberate fault or gross negligence or in the case of mildly negligent injury to life or limb.

Weinmann reserves the right to eliminate the fault, to supply a non-defective item or to reduce the purchase price appropriately at its discretion.

If a claim under warranty is rejected, we will not bear the costs of transport to and from ourselves.

Statutory claims under warranty remain unaffected.

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## 1. Description of device

### Key

- 1 Locking key
- 2 Top part of housing
- 3 Pressure port
- 4 Sealing element
- 5 Bottom part of housing
- 6 Outlet connection port
- 7 Particle filter
- 8 Drying adapter
- 9 Therapy device
- 10 Respiratory air humidifier
- 11 Arrows for connecting function

### 1.1 Intended purpose

The bacteria filter serves to filter respiratory air on therapy devices for sleep-related respiratory disorders and ventilatory insufficiency to prevent devices becoming contaminated. The maximum duration of use for the particle filter is 24 h.

### 1.2 Description of function

The bacteria filter is located between the device and the tube system. This includes a particle filter which filters the patient's respiratory air. The filtering action of the particle filter is achieved by means of electrostatic charging.

## 2. Safety information

### 2.1 Safety regulations

For your own safety and for that of your patients and to satisfy the requirements of the German law governing medical products [Medizinproduktegesetz - MPG], please note the following:

- Please read these instructions for use carefully. They are a constituent part of the device and must be available at all times.
- Use the bacteria filter only for the intended purpose described (see 1.1 "Intended purpose").
- For each new patient, a new particle filter should always be used and the housing disinfected/sterilized both inside and out. Please see section entitled „5. Hygienic preparation“ to avoid an infection or bacterial contamination.
- Malfunctions and a lack of biocompatibility may result if third-party articles are used. Please bear in mind that in these cases any warranty entitlement and liability shall lapse where the accessories recommended in the instructions for use or original spare parts are not utilised.

### 2.2 Special marks on the device

-  Connecting position
-  Device inlet
-  Device outlet

#### Particle filter only

-  Disposable item

## 3. Setting up and assembling the device

### 3.1 Fitting the bacteria filter

A particle filter 7 must be inserted before use. Please proceed as outlined below.

1. Unscrew the two halves of the housing by rotating to the left.
2. If necessary, remove old particle filter 7 from bottom part of the housing 5.
3. Insert new particle filter 7. Ensure that particle filter 7 lies evenly on the ribs.
4. Push the two halves of the housing together so that the arrows for connecting position 11 are opposite one another. Then screw the halves of the housing up to the stop. In the final position, the webs for the handles of the two halves of the housing are opposite one another.

### 3.2 Combining with other devices

You must follow the instructions for use for the combined devices.

#### Therapy devices

The bacteria filter can be combined with the following therapy devices:  
SOMNOcomfort®, SOMNOvent® S, SOMNOvent® ST, SOMNOsmart® 2, SOMNOset® and VENTImotion®

#### Tube system

The bacteria filter can be combined with the following tube systems:

- tube system (can be disinfected) WM 24130
- tube system (can be sterilized) WM 24120

#### Respiratory air humidifier

The bacteria filter can be combined with the SOMNOclick® and SOMNOclick® 300 respiratory air humidifiers.

### 3.3 Connecting the bacteria filter

1. If necessary, disconnect the corrugated tube from the therapy device by pressing the locking key of the tube adapter.
2. Place the bacteria filter on the therapy device outlet or on the humidifier (refer to Legend). Ensure that locking device 1 engages.
3. Click the adapter of the corrugated tube into outlet connection part 6 of the bacteria filter.

## 4. Operation

### 4.1 Starting up

1. Start up the device combination in accordance with the instructions for use for the therapy device.

### 4.2. After use

1. Switch off the therapy device.
2. Take off the corrugated tube by pressing the locking key.
3. Take off the bacteria filter by pressing locking key 1.
4. Clean the bacteria filter as described under „5. Cleaning, disinfecting and sterilizing“.

#### Note!

To avoid contamination, always use a new particle filter after:

- max. 24 h use
- opening the bacteria filter for cleaning purposes

## 5. Hygienic preparation

### 5.1. Critical times

- Particle filter 7 must be changed after 24 h (3 nights of 8 h each).
- The housing of the bacteria filter should be cleaned after 24 h (3 nights of 8 h each).
- Particle filter 7 must always be changed and the housing disinfected or sterilized whenever there is a new patient.
- The pressure measuring tube of the tube system should be changed every four weeks on hygiene grounds.

### 5.2. Cleaning

1. Unscrew the two halves of the housing by rotating them to the left.
2. Remove particle filter 7.
3. Clean the parts of the housing in a dishwasher at 65°C, using the top basket of a dishwasher and positioning the housing so that all the areas of the top and bottom part can be thoroughly washed.
4. Rinse the parts of the housing afterwards in clean water.
5. Shake the parts of the housing vigorously to remove water.
6. Push the two halves of the housing together so that the arrows for connecting position 11 are opposite one another. Then screw the halves of the housing up to the stop. In the final position, the webs for the handles of the two halves of the housing are opposite one another.
7. Plug the red drying adapter 8 (included in the scope of supply of the therapy device) onto the therapy device.
8. Plug the bacteria filter onto drying adapter 8.
9. Activate drying mode by pressing the Softstart (Symbol) and On/Off (Symbol) keys simultaneously (see "Cleaning and disinfecting" in the instructions for use for the therapy device). The device now switches on and dries the pressure measuring section. The time remaining is displayed. The device switches off automatically after 30 minutes.
10. Press locking key 1 and take the bacteria filter off drying adapter 8.
11. Take drying adapter 8 off your therapy device by twisting slightly.

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WEINMANN

## EEC Letter of Conformity on Medical Products

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declare in sole  
responsibility that

Product Name: **Breathing system filter**  
Type/Model: **Bacteria filter**

being in conformity  
with the respective  
regulations of the  
following guideline:

Directive 93/42/EEC on Medical Products

Classification:

IIa

Standards used  
in particular:

DIN EN ISO 17510-1:2000  
DIN EN 13328-1 04/02 p6EN 13328-2 02/01

In Hamburg on July 24, 2002,  
and amended on January 14, 2003

  
Dr. J. G. G. G.  
Executive President  
Chief Executive Officer

  
Dr. K. A. F. F.  
Executive Vice President  
Chief Operating Officer

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Geräte für Medizin und Arbeitsschutz GmbH + Co.