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Legende

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1 Device Description

Key

- Forehead cushion
- Mask seal
- Head strap
- Rotating sleeve
- Support
- Seal for ports
- Mask body
- Elbow
- Exhalation connector
- Exhalation slot
- Hose connector

Purpose of Use

The nasal mask with exhalation connector is an accessory that serves as a sealing element between patient and therapy device. It is used in the treatment of sleep apnea and for non-invasive ventilation of patients with respiratory insufficiency.

Functional Description

The nasal mask and the head strap are attached over your nose so that the flow of air from the therapy device is routed through the nasal mask to the nose. The form and flexible material of the mask seal creates an airtight band to the face. The elbow connector allows the greatest possible freedom in positioning the hose.

The inhaled air flows through the mask/hose system. The exhaled air escapes through a slot between the elbow and the rotating sleeve. This allows you to breathe freely during therapy. The exhalation connector avoids the risk of re-inhalation.

2 For Your Safety

Safety regulations

For your and your patients' safety and in accordance with the requirements of the European Union Guidelines 93/42/EEC, please observe the following points:

- Please read these instructions for use very carefully. They are a part of the mask and should be available at all times.
- Malfunctions and a lack of biocompatibility may result if third-party articles are used. Please bear in mind that in these cases any warranty entitlement and liability shall lapse where the accessories recommended in the instructions for use or original spare parts are not utilised.
- The mask is designed for therapy pressure between 4 and 30 hPa. The nasal mask must not be used at higher or lower therapy pressures.

- Note that patients with limited spontaneous respiration must be monitored at all times and that the appropriate negative pressure/leak alarm on the ventilation device must be activated.
- Patients with obstructive and restrictive lung diseases should be examined on an individual basis during use of the SOMNOplus®, for example with regard to re-inhalation of CO₂.
- The SOMNOplus® has an integrated exhalation system. Do not connect any other exhalation system or exhalation valve to the nasal mask.
- Do not block any of the openings in the nasal mask.
- Only put the nasal mask over your face for a prolonged period when the therapy device is switched on. The flow of air from the therapy device serves among other things to remove the exhaled air from mask and hose. If the therapy device is not switched on, there is a risk of re-inhalation when you breathe through your nose.
- Do not use a breathing hose that is electrically conductive.
- Clean the mask as described under Section 4 "Hygienic preparation". Inappropriate cleaning procedures, unsuitable cleaning agents and infrequent cleaning can result in an infection or bacterial contamination and can impede the functioning of the mask.

Side effects

The following side effects may occur during short-term or long-term use: nasal congestion, dry nose, dry mouth in the morning, sinus pressure, irritation of the conjunctiva as well as rattling noises during exhalation. Wearing the mask can also lead to tenderness of the face or reddening of the skin.

3 Operation

Assemble and adjust forehead support

The forehead support is made up of a support and a forehead cushion. The support makes it possible to adjust the mask to your face while the forehead cushion allows for comfortable distribution of the mask pressure across the forehead. By adjusting the support, pressure on the bridge of the nose can be minimized (Fig. A).

Select the support size that best suits your face (Fig. B). Attach the forehead cushion to the support and place the support in the slit on the mask body. In most cases, the size 3 support will be appropriate. The forehead cushion is already set up on this support when you receive the mask.

Place the mask on your face and check to see if you have chosen the correct support.

If the mask isn't tight or if it causes an uncomfortable tenderness, shift the support within the slot of the mask or choose a different support size.

Dismantle forehead support

Loosen the forehead support by pressing on the clamp on the mask body (Fig. C). Pull forehead cushion off support.

Head strap adjustment

When you first put on the nasal mask with the head straps, it is a good idea to ask another person for help.

- Thread the four Velcro straps of the head strap into the slots on the mask body. You can use either the lateral slots or the top hook to attach the Velcro forehead straps. This is not necessary the first time the nasal mask is used.
- Pass the head strap over your head and hold the nasal mask over your nose.
- Adjust the head and neck straps of the head strap so that the mask sits firmly, but not too tightly, against your face (Fig. D).

Connection to therapy device

Push the hose of your therapy device onto the rotating sleeve (Fig. E). In doing so, ensure that there is a secure, airtight connection to the hose. The connection should not come loose as a result of therapy pressure.

If you wish to interrupt therapy briefly, you can pull the rotating sleeve out of the elbow. It is not necessary to remove the mask or pull the breathing hose off the rotating sleeve with this nasal mask.

Combining with other devices

This nasal mask can be combined with any device to perform the therapies described under "Purpose of use", as long as the following conditions are met:

- when combining the nasal mask with other devices, be aware that the therapy pressure in the nasal mask depends on the combination of devices. Ask your doctor or dealer whether the correct therapy pressure can be achieved in the mask using the desired combination of devices. Have the settings of your therapy device checked by your doctor or dealer.
- take account of the specifications in the instructions for use for the devices you intend combining with the nasal mask.

Use with cold-air or warm-air humidifier

The nasal mask system can be used with a cold-air or warm-air humidifier. The degree of humidity you choose should not be so high that water condenses in the air hose. Please be sure to read the instructions for use for your humidifier and your therapy device.

Pressure measurement

SOMNOplus® is supplied with two ports. They can be used for external pressure measurement.

If only one port is required, turn the connection upwards (Fig. F). When the functions with the ports are no longer required, they should be made airtight with the enclosed seal. Otherwise air can escape through the ports and therapy pressure will not be maintained.

After use

- Undo one Velcro strap on the head strap and remove the mask.
- Clean the mask as described in Section 4.

Dismantling the mask

- Remove the head strap from the nasal mask.
- Loosen the forehead support from the mask body by pressing on the clamp. Pull the forehead cushion off the support.
- Pull the mask seal off the mask body and remove the seal for the ports.
- Pull the rotating sleeve out of the elbow and remove the elbow from the mask body by pressing gently inwards (Fig. G).

Assembling the mask

- Push the elbow into the mask body from inside. Push the rotating sleeve into the elbow. Make sure both parts engage properly.
- Push the mask seal onto the mask body and the port seal onto the pressure measuring port.
- Attach the forehead support to the mask body as described under "Assemble and adjust forehead support".
- Attach the head strap to the nasal mask as described under "Head strap adjustment".

4 Hygienic preparation

4.1 Schedule

Daily cleaning

Please clean the nasal mask after every usage as described in item 4.2 "Procedure under "Cleaning by hand". For cleaning purposes we recommend taking the mask apart as described above.

Weekly cleaning, disinfecting or sterilizing

The nasal mask should be cleaned thoroughly at least once per week. For cleaning purposes, take the mask apart as described above.

4.2 Procedure for cleaning, disinfecting or sterilizing

The following procedures are allowed:

Parts	Procedures	Cleaning by hand	Dishwasher	Washing machine	Disinfecting*	Sterilizing (as an alternative to disinfecting)
Mask seal	X	X	X	X	X	X
Forehead cushion	X	X	X	X	X	X
Seal for ports	X	X	X	X	X	X
Mask body	X	X	X	X	X	X
Support	X	X	X	X	X	X
Elbow	X	X	X	X	X	X
Rotating sleeve	X	X	X	X	X	X
Head strap	X	X	X	X	X	X

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Cleaning by hand	Clean in warm water, free of residues with a mild detergent as for dishwashing. Carefully rinse the parts with clear water.
Dishwasher	Wash at max. 65°C. Carefully rinse the parts with clear water.
Washing machine	Wash at 30°C.
Disinfection at home (no change in parts!)	Clean all parts as described under "Cleaning by hand" and then immerse them in a diluted solution of disinfectant. After disinfecting rinse all parts thoroughly with clear water.
Disinfecting at clinics and with change in patients*	Clean in a diluted solution using mechanical means, e.g. with a brush. After disinfecting, rinse the parts thoroughly with distilled water and dry them completely before you reassemble the mask.
Sterilizing (as an alternative to disinfecting)	Steamsterilize in devices complying with EN 285. Temperature 134 °C, Minimum time: 3 minutes.

Do all parts thoroughly after cleaning. Check components for residual dirt or damage. Repeat cleaning if necessary. If parts are damaged, replace them. Assemble the nasal mask as described above.

*** We recommend the disinfectant Gigasept FF. Make sure that you follow the manufacturer's directions. Tolerance of this disinfectant is proven. We recommend that you wear suitable gloves when disinfecting the equipment (e.g. household or disposable gloves).**

4.3 Instructions for Disinfecting and Sterilizing

For patients with a weakened immune system or a special medical history daily disinfection or sterilisation may be necessary as for a change in patients. In this case, please consult your doctor.

4.4 Disinfecting and Sterilizing for new patient

If the nasal mask is to be used for another patient, it must be subjected to hy-giene measures first. Disassemble the mask and disinfect or sterilize all parts as described above or use a new mask. Use a new head strap.

5 Functional checks

Please carry out a visual check once per week. If the mask seal is no longer airtight due to a tear, please change it. Please check the functional capacity and airtightness of the ball-and-socket joint. If it no longer moves easily or is no longer airtight, please change the elbow connection and/or the body of the mask. Parts that can no longer be used can be disposed of normal household waste.

6 Troubleshooting

Fault	Cause	Remedy
In the morning you have pain from pressure on your face.	The nasal mask is too tight.	Enlarge the headgear or the head straps a bit. Change the position of the forehead support on the mask body. If necessary, select a different support size for the forehead cushion.
Your eyes feel a draught.	The mask is too loose	Make the headgear or the straps a little tighter. Select a different support size for the forehead cushion.
The mask doesn't fit.	Please see if you need a different size mask. Your specialist dealer can advise you.	
The mask is not airtight.	Correct the fit, adjust the head straps. Change the position of the forehead support on the mask body, if necessary, select a different support size for the forehead cushion.	
Therapy pressure is not reached.	Leaks in the shaped mask. Supply system is not fitted. Mask blows air over the open ports.	Change the mask seal. Check the plug connection and the fit of the hose muff. Place the seal on the ports.

7 Supply Schedule

Supply Schedule	SOMNOplus®, size S	WM 25710
consisting of:	Mask seal size S	WM 25111
	Body of mask size S	WM 25112
	Seal for parts	WM 25165
	Forehead support, sizes 1-4 (size 3 fitted)	WM 25180
	Exhalation connector	WM 25705
	consisting of elbow and rotating sleeve SOMNOstrap® (blue)	WM 23695
	Instructions for use	
	SOMNOplus®, size M	WM 25720
	Like the SOMNOplus®, size S, but with Mask seal size M	WM 25121
	Body of mask size M	WM 25122

SOMNOplus®, size L	WM 25730
Like the SOMNOplus®, size S, but with Mask seal size L	WM 25131
Body of mask size L	WM 25132

Replacement parts	Forehead cushion	WM 15500
	Forehead support size 1	WM 15592
	Forehead support size 2	WM 15593
	Forehead support size 3	WM 15594
	Forehead support size 4	WM 15595
	Instructions for use DE – GB – FR – IT – NL	WM 16806
	Instructions for use ES – NO – SE – DK – FI	WM 16807

All parts in the Supply Schedule are also available as individual parts.

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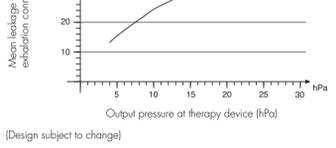
Device class according to Directive européenne CE 93/42 : II a	Dimensions (WxHxD): approx. 80 x 140 x 90 mm
Weight: approx. 55 grammes	
Therapy pressure range: 4 to 30 hPa	
Hose connector: Cone 22 mm diam. (EN ISO 5356-1) for elastomeric hoses	
Port pressure measurement: Ø 4 mm	
Connection head straps: Ø 4 mm	
Operating temperature range: + 5° C to + 40° C	
Storage temperature range: - 20° C to + 70° C	
Flow resistance at 50 l/min: 0,06 hPa	
Flow resistance at 100 l/min: 0,22 hPa	
Noise level at 10 hPa: 29 dB(A)	
Duration of use: 6 to 12 month depending on frequency of use and cleaning	

Materials	Silicone
- Mask seal	polycarbonate
- Mask body	polycarbonate
- Elbow	polycarbonate
- Rotating sleeve	polycarbonate
- Seal for parts	silicone
- Forehead cushion	silicone
- Support	polypropylene
- SOMNOstrap®	cotton, polyamide

All individual parts of the nasal mask are latex-free.

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(Design subject to change)

9 Warranty

- Weinmann guarantees that the product, when used in accordance with requirements, will remain free from defects for a period of 6 months from date of purchase.
- Claims against the warranty can be made only when accompanied by the sales receipt, which must show salesperson and date of purchase.
- We do not offer a warranty if:
 - the instructions for use are not followed
 - operation is faulty
 - the device is used or handled incorrectly
 - an unauthorized third party interferes with the device for the purposes of repair
 - an act of God is involved
 - transport damage is sustained as a result of returned items not being packed correctly
 - wear and tear incurred by normal use is involved
 - failure to use original spare parts.
- Weinmann is not liable for consequential harm caused by a defect if it is not due to deliberate or gross negligence. Weinmann is also not liable for minor physical injury to life or limb resulting from negligence.
- Weinmann reserves the right to decide whether to eliminate defects, to deliver a defect-free item or to reduce the purchase price by a reasonable amount.
- If Weinmann rejects a claim against the warranty, it assumes no expense for transport between customer and manufacturer.
- Implied rights to claim under warranty remain unaffected by these changes.

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