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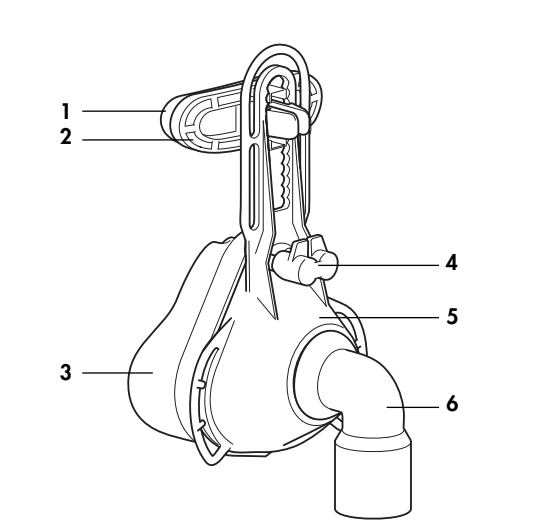
Legende

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1 Device Description

Key

- Forehead cushion
- Support
- Mask seal
- Seal for ports
- Mask body
- Elbow connection

Purpose of Use

The nasal mask is an accessory that serves as a sealing element between patient and therapy device. It is used in the treatment of sleep apnea and for non-invasive ventilation of patients with respiratory insufficiency.

Functional Description

The air flow from your therapy device is directed toward the nose via the nasal mask.

The form and flexible material of the mask seal creates an airtight bond to the face. The elbow connection allows the greatest possible freedom in positioning the hose.

Also necessary for therapy:

- an exhalation system (e.g., a noise suppressor or Silentflow) through which exhalation takes place. The risk of rebreathing is avoided through use of such a system.
- headgear or straps for attachment to the head.

2 For Your Safety

Safety regulations

For your and your patients' safety and in accordance with the requirements of the European Union Guidelines 93/42/EEC, please observe the following points:

- Please read these instructions for use very carefully. They are a part of the mask and should be available at all times.
- Malfunctions and a lack of biocompatibility may result if third-party articles are used. Please bear in mind that in these cases any warranty entitlement and liability shall lapse where the accessories recommended in the instructions for use or original spare parts are not utilised.
- The mask is designed for therapy pressure between 4 and 30 hPa. The nasal mask must not be used at higher or lower therapy pressures.
- Note that patients with limited spontaneous respiration must be monitored at all times and that the appropriate negative pressure/leak alarm on the ventilation device must be activated.
- The connection to the device's hose system must be airtight and should not be allowed to loosen due to therapy pressure.
- Do not use a breathing hose that is electrically conductive.

- Clean the mask as described under Section 4 "Hygienic preparation". Inappropriate cleaning procedures, unsuitable cleaning agents and infrequent cleaning can result in an infection or bacterial contamination and can impede the functioning of the mask.
- These nasal masks can be used with all devices for treatment described under "Purpose of Use" if the following conditions are fulfilled:
 - When using the nasal mask with other devices, make sure that the therapy pressure in the nasal mask is in accordance with the device combination. Ask your doctor or authorized dealer whether the correct therapy device in the mask can be achieved with the selected device combination. Have your doctor or dealer check the settings on your therapy device.
 - Observe the instructions in the User's Manual for the devices you use with your nasal mask.
 - Use an exhalation system with a connection in accordance with DIN EN 1281-1, such as a noise suppressor or Silentflow.

Side effects

The following side effects may occur during short-term or long-term use:

nasal congestion, dry nose, dry mouth in the morning, sinus pressure, irritation of the conjunctiva as well as irritating noises during exhalation. Wearing the mask can also lead to tenderness of the face or reddening of the skin.

3 Operation

Assemble and adjust forehead support

The forehead support is made up of a support and a forehead cushion. The support allows for comfortable distribution of the mask pressure across the forehead. By adjusting the support, pressure on the bridge of the nose can be minimized.

Select the support size that best suits your face. Attach the forehead cushion to the support and place the support in the slit on the mask body. In most cases, the size 3 support will be appropriate. The forehead cushion is already set up on this support when you receive the mask.

To fasten the forehead cushion, we recommend securing one end of the cushion to the support first. Then, while holding that end tight, pull the other end over the opposite end of the support and attach it to the support.

Place the mask on your face and check to see if you have chosen the correct support.

If the mask can't tight or if it causes uncomfortable tenderness, shift the support within the slit of the mask or choose a different support size.

Dismantle forehead support

Loosen the forehead support by pressing on the clamp on the mask body. Pull forehead cushion off support.

Put on headgear

When you first put on the nasal mask with the headgear, it is a good idea to ask another person for help.

- Thread both Velcro forehead straps on the headgear through the cutouts on the forehead support.
- Thread one of the Velcro chin straps through the side slot and put on the mask.
- Lastly, thread the other Velcro chin strap through the slot and adjust the straps for an even, comfortable and tight fit on your face.

You can use the side slots or the upper curve to fasten the mask.

Head strap adjustment

When you first put on the nasal mask with the head straps, it is a good idea to ask another person for help.

- When putting on the head straps, proceed as described under "Put on headgear".

An exact individual adjustment is possible with the neck strap and the head-band.

Exhalation device installation

Attach the exhalation device to the hose.

Connection to therapy device

Connect the exhalation device to the mask.

It should not be possible to loosen the connection with a normal pull or with therapy pressure.

Use with cold-air or warm-air humidifier

The nasal mask system can be used with a cold-air or warm-air humidifier. The degree of humidity you choose should not be so high that water condenses in the air hose. Please be sure to read the instructions for use for your humidifier and your therapy device.

Pressure measurement

SOMNOmask® is supplied with two ports. They can be used for external pressure measurement.

If only one port is required, turn the connection upwards (Fig. F). When the functions with the ports are no longer required, they should be made airtight with the enclosed seal. Otherwise air can escape through the ports and there-fore pressure will not be maintained.

After use

- Loosen one of the Velcro chin straps and take off the mask.
- Disconnect the exhalation device from the mask.
- Clean the mask as described in Section 4.

For information regarding cleaning of the exhalation device and the head-gear, please observe the manufacturers instructions.

Dismantling the mask

- Remove the exhalation device and headgear from the nasal mask.
- Loosen the forehead support from the mask body by pressing on the clamp. Pull the forehead cushion from the support.
- Pull the mask seal from the mask body and remove the seal for the ports if necessary.
- Remove the elbow connection from the body of the mask by pressing it lightly.

To assemble the mask, please follow the instructions in the reverse direction.

4 Hygienic preparation

4.1 Schedule

Daily cleaning

Please clean the nasal mask after every usage as described in item 4.2 "Pro-cedure" under "Cleaning by hand". For cleaning purposes we recommend tak-ing the mask apart as described above.

Weekly cleaning, disinfecting or sterilizing

The nasal mask should be cleaned thoroughly at least once per week. For cleaning purposes, take the mask apart as described above.

4.2 Procedure for cleaning, disinfecting or sterilizing

The following procedures are allowed:

Procedures					
Parts	Cleaning by hand	Disinfectant	Disinfecting*	Sterilizing (as an alternative to disinfecting)	
Mask seal	X	X	X	X	
Forehead cushion	X	X	X	X	
Seal for ports	X	X	X	X	
Mask body	X	X	X		
Support	X	X	X		
Elbow connection	X	X	X		

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Cleaning by hand	Clean in warm water, free of residues with a mild detergent as for dishwashing. Carefully rinse the parts with clear water.
Dishwasher	Wash at max. 65°C. Carefully rinse the parts with clear water.
Disinfection at home (no change in patients!)	Clean all parts as described under "Cleaning by hand" and then immerse them in a diluted solution of disinfectant. After disinfecting rinse all parts thoroughly with clear water.
Disinfecting at clinics and with change in patients*	Clean in a diluted solution using mechanical means, e.g. with a brush. After disinfecting, rinse the parts thoroughly with distilled water and dry them completely before you reassemble the mask.
Sterilizing (as an alternative to disinfecting)	Steamsterilize in devices complying with EN 285, Temperature 134 °C, Minimum time: 3 minutes
Dry all parts thoroughly after cleaning. Check components for residual dirt or damage. Repeat cleaning if necessary. If parts are damaged, replace them. Assemble the nasal mask as described above.	

• We recommend the disinfectant Gigasept FF. Make sure that you follow the manufacturer's directions. Tolerance of this disinfectant is proven. You are recommended to wear suitable gloves (e.g. household or disposable gloves) during disinfection procedures.

4.3 Instructions for Disinfecting and Sterilizing

For patients with a weakened immune system or a special medical history daily disinfection or sterilisation may be necessary as for a change in patients. In this case, please consult your doctor.

4.4 Disinfecting and Sterilizing for new patient

If the nasal mask is to be used for another patient, it must be subjected to hygienic measures first. Dismantle the mask and disinfect or sterilize all parts as described above or use a new mask.

5 Functional checks

Please carry out a visual check once per week. If the mask seal is no longer airtight due to a tear, please change it. Please check the functional capacity and airtightness of the ball-and-socket joint. If it no longer moves easily or is no longer airtight, please change the elbow connection and/or the body of the mask.

Parts that can no longer be used can be disposed of normal household waste.

6 Troubleshooting

In the morning you have pain from pressure on your face.

Your eyes feel a draught.

The exhalation device loosens itself from the mask.

Therapy pressure is not reached.

Tears in the shaped mask.

Supply system is not airtight.

Mask blows air over the open ports.

Place the seal on the ports.

Enlarge the headgear or the head straps a bit. Change the position of the forehead support on the mask body. If necessary, select a different support size for the forehead cushion.

Make the headgear or the straps a little tighter. Select a different support size for the forehead cushion.

Please see if you need a different size mask. Your specialist dealer can advise you.

The connection of your exhalation device does not correspond to the norm. Ask your specialty dealer to check the connection.

Correct the fit, adjust the head straps. Change the position of the forehead support on the mask body. If necessary, select a different support size for the forehead cushion.

Change the mask seal.

Check the plug connection and the fit of the hose muff.

Place the seal on the ports.

7 Supply Schedule

Supply Schedule	transparent	blue
SOMNOmask [®] , size S consisting of:	WM 25110	WM 25210
Mask seal size S	WM 25111	
Body of mask size S	WM 25112	WM 25212
Elbow connection	WM 25145	WM 25245
Seal for ports	WM 25165	
Forehead support, sizes 1-4	WM 25180	WM 25280
SOMNOmask [®] , size M consisting of:	WM 25120	WM 25220
Mask seal size M	WM 25121	

Body of mask size M	WM 25122	WM 25222
Elbow connection	WM 25145	WM 25245
Seal for ports	WM 25165	
Forehead support, sizes 1-4	WM 25180	WM 25280
SOMNOmask [®] , size L consisting of:	WM 25130	WM 25230
Mask seal size L	WM 25131	
Body of mask size L	WM 25132	WM 25232
Elbow connection	WM 25145	WM 25245
Seal for ports	WM 25165	
Forehead support, sizes 1-4	WM 25180	WM 25280

Replacement parts	
Cushion frontal	WM 15500
Support frontal Taille 1	WM 15592
Support frontal Taille 2	WM 15593
Support frontal Taille 3	WM 15594
Support frontal Taille 4	WM 15595
Support frontal Taille 1 (blue)	WM 15596
Support frontal Taille 2 (blue)	WM 15597
Support frontal Taille 3 (blue)	WM 15598
Support frontal Taille 4 (blue)	WM 15599
All parts in the Supply Schedule are also available as individual parts.	

Other available products	
Exhalation devices:	
Noise suppressor	WM 23685
Silentflow 2	WM 23600
Headgear:	
4-point headgear size M (white)	WM 23686
4-point headgear size L (white)	WM 23687
4-point headgear size M (blue)	WM 23673
4-point headgear size L (blue)	WM 23674
Head straps:	
SOMNOstrap [®] (white)	WM 23177
SOMNOstrap [®] (blue)	WM 23695

8 Technical Data	
Device class acc. to 93/42 EEC:	II a
Dimensions (WxHxD):	approx. 80 x 140 x 90 mm
Weight:	approx. 50 grammes
Pressure range:	4 to 30 hPa
Hose connector:	22 mm conical connector in accordance with DIN EN 1281-1
Port pressure measurement:	Ø 4 mm
Connection head straps:	Strap width max. 28 mm
Operating temperature range:	+ 5° C to + 40° C
Storage temperature range:	- 20° C to + 70° C
Flow resistance at 50 l/min	0,25 hPa

Flow resistance at 100 l/min	0,92 hPa
Duration of use:	6 months, with careful maintenance up to 12 months, depending on frequency of use and cleaning

Materials	
- Mask body	PC (polycarbonate)
- Mask seal	LSR (silicon)
- Elbow connection	PC (polycarbonate)
- Seal for ports	LSR (silicon)
- Forehead cushion	LSR (silicon)
- Support	PC (polycarbonate)
All individual parts of the nasal mask are latex-free.	
Symbols used:	
☉ Storage temperature	
(Design subject to change)	

9 Warranty

Weimann guarantees that the product, when used in accordance with requirements, will remain free from defects for a period of 6 months from date of purchase.

Claims against the warranty can be made only when accompanied by the sales receipt, which must show salesperson and date of purchase.

We do not offer a warranty if:

- the instructions for use are not followed

- operation is faulty

- the device is used or handled incorrectly

- an unauthorized third party interferes with the device for the purposes of repair

- an act of God is involved

- transport damage is sustained as a result of returned items not being packed correctly

- wear and tear incurred by normal use is involved

- failure to use original spare parts.

Weimann is not liable for consequential harm caused by a defect if it is not due to deliberate or gross negligence. Weimann is also not liable for minor physical injury to life or limb resulting from negligence.

Weimann reserves the right to decide whether to eliminate defects, to deliver a defect-free item or to reduce the purchase price by a reasonable amount.

If Weimann rejects a claim against the warranty, it assumes no expense for transport between customer and manufacturer.

Implied rights to claim under warranty remain unaffected by these changes.

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